


Facial muscles

1. Frontalis
2. Corrugator supercilii
3. Procerus
4. Depressor supercilii
5. Orbicularis oculi (superior lateral)
6. Orbicularis oculi (lateral)
7. Nasalis
8. Levator labii superioris alaeque nasi
9. Levator labii superioris
10. Zygomaticus minor
11. Zygomaticus major
12. Orbicularis oris
13. Buccinator
14. Risorius
15. Masseter
16. Depressor anguli oris
17. Depressor labii inferioris
18. Platysma
19. Mentalis

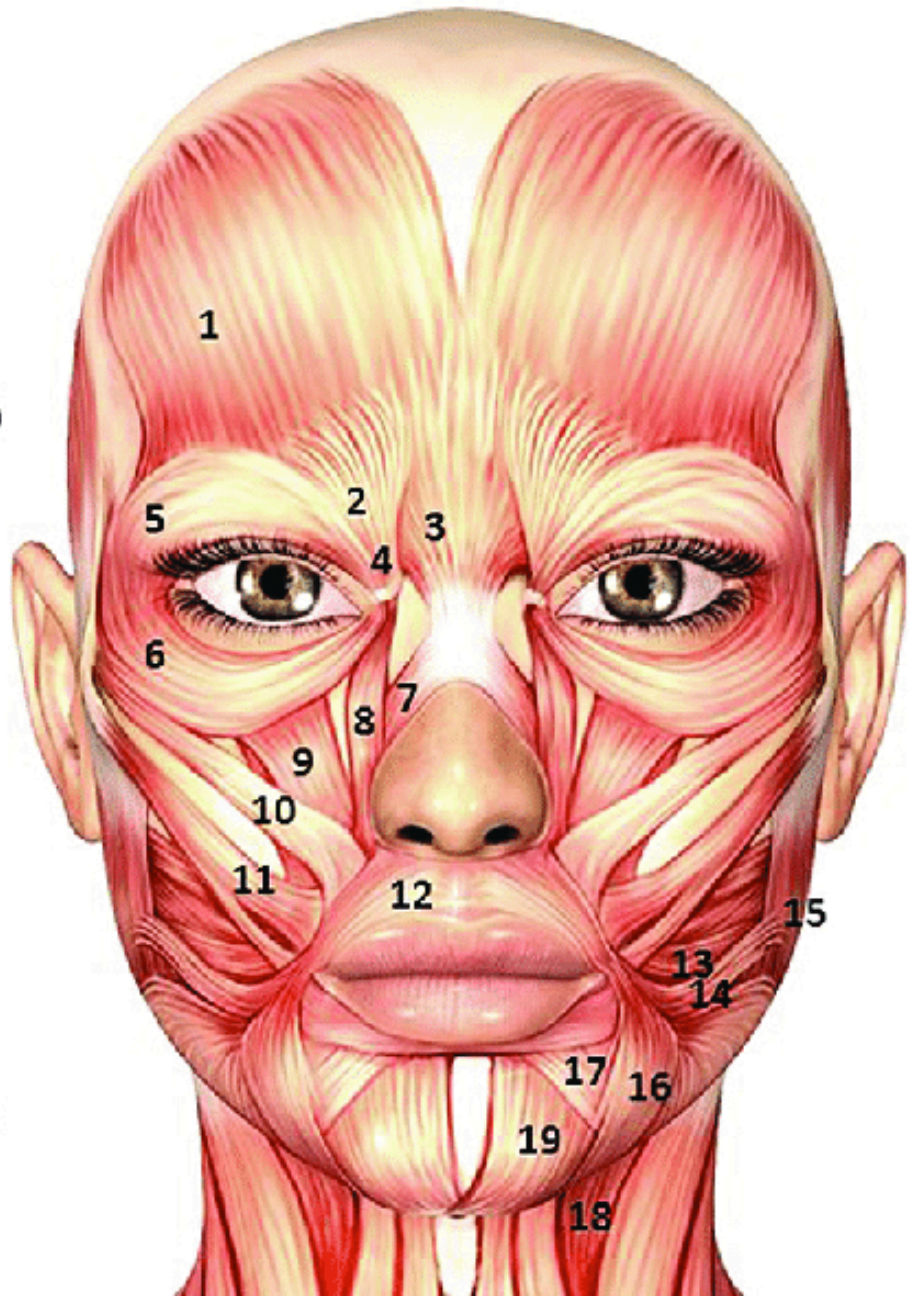

Step 1.

Turn on the device and wait a few seconds.



Step 2.

Click the top button twice to set the intensity.


2x



Step 3.

- Take enough gel and place the Biorefpen on the lower left side of the mouth.
- A red light will now illuminate.
- Go 1x all over the face.



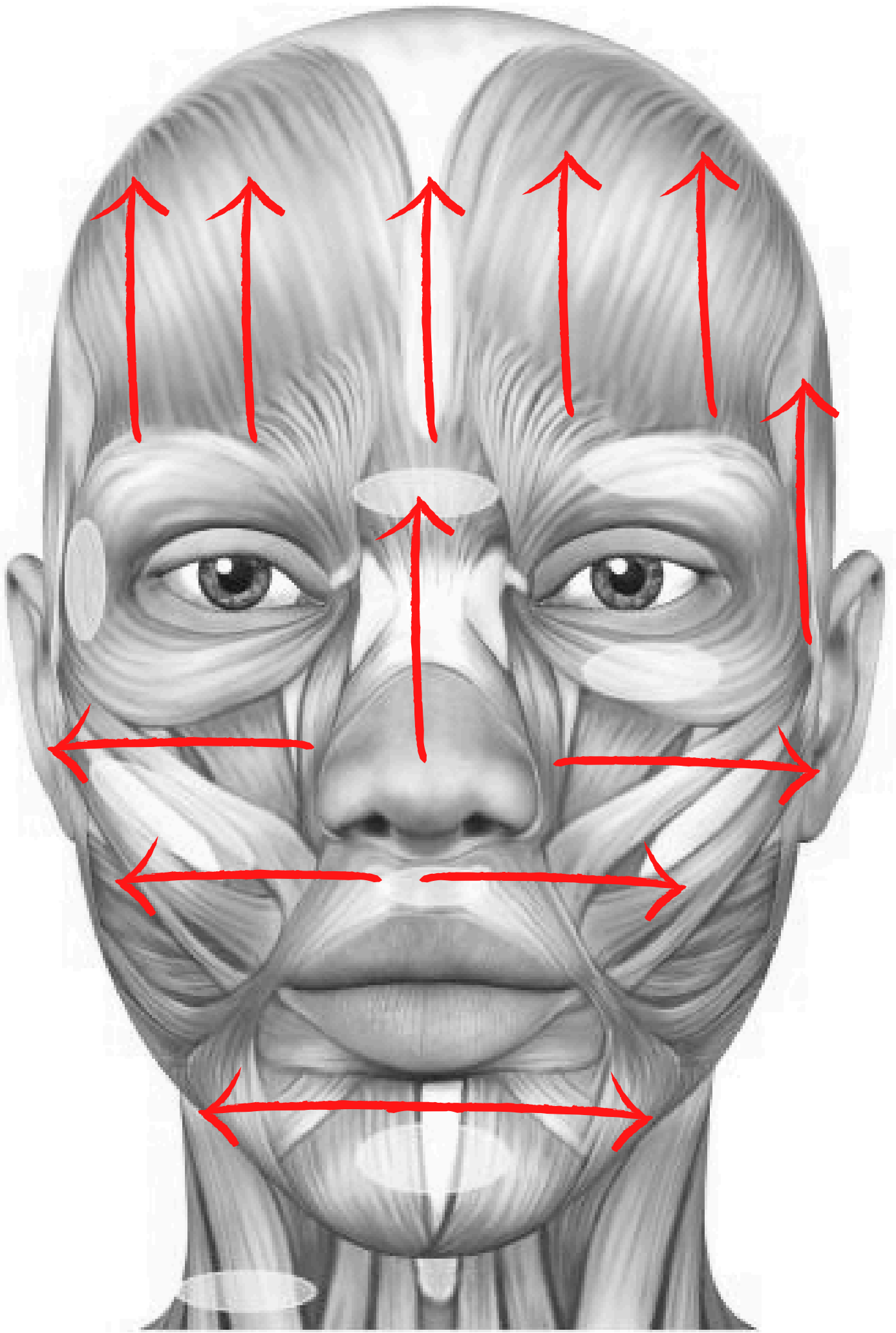


Step 4.

- Now click the bottom button twice to switch to alternate mode.
- Take enough gel and put the Biorefen at the bottom of the chin.
- A green and red light will now turn on alternately.
- We now follow the different trajectories from attachment to attachment. See drawing.
- Always wait three sec at each attachment point and repeat each passage three times.
- You take the important zones an additional three or six times.




Passage 1.


✗ = 3 sec wait.

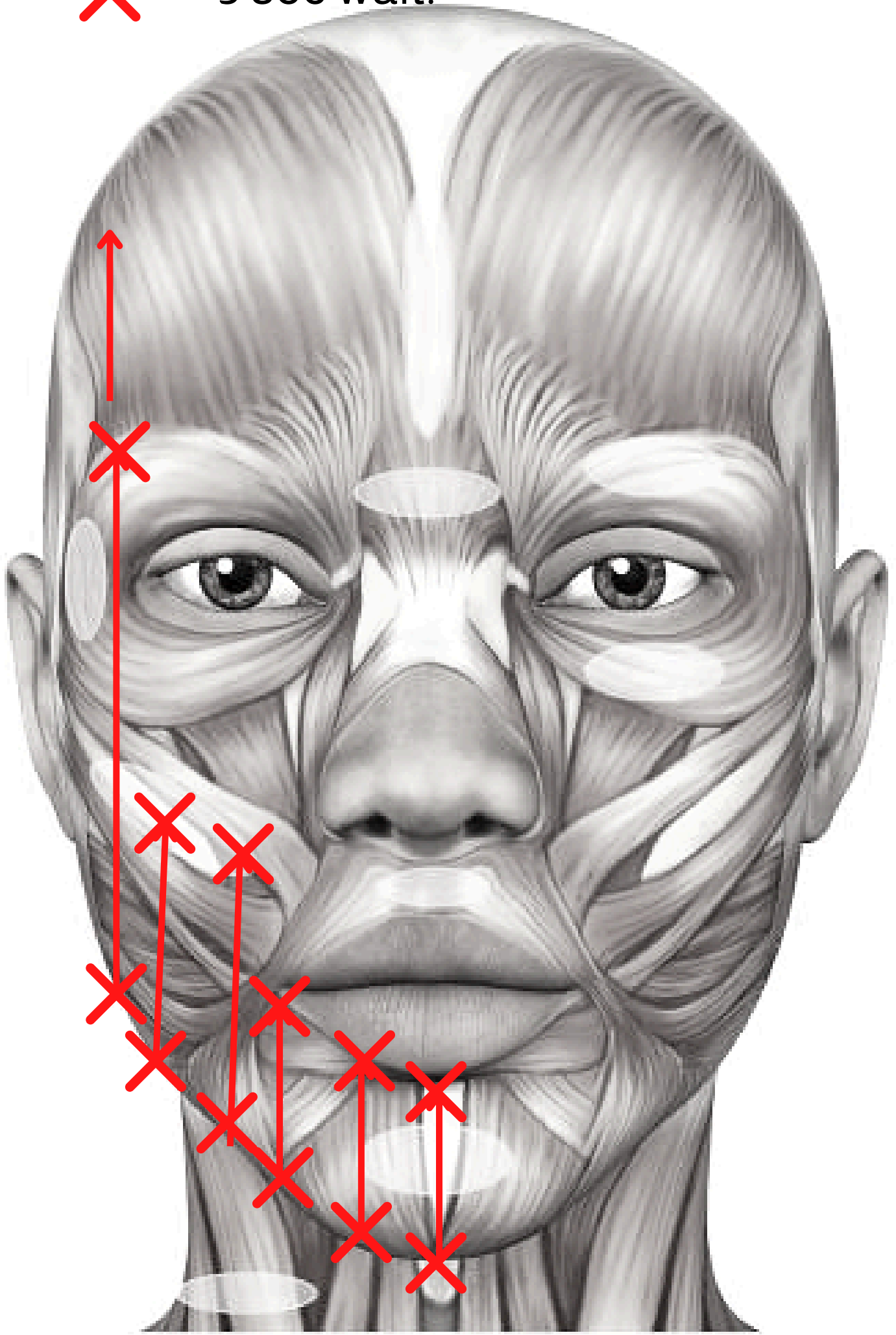


Passage 2.

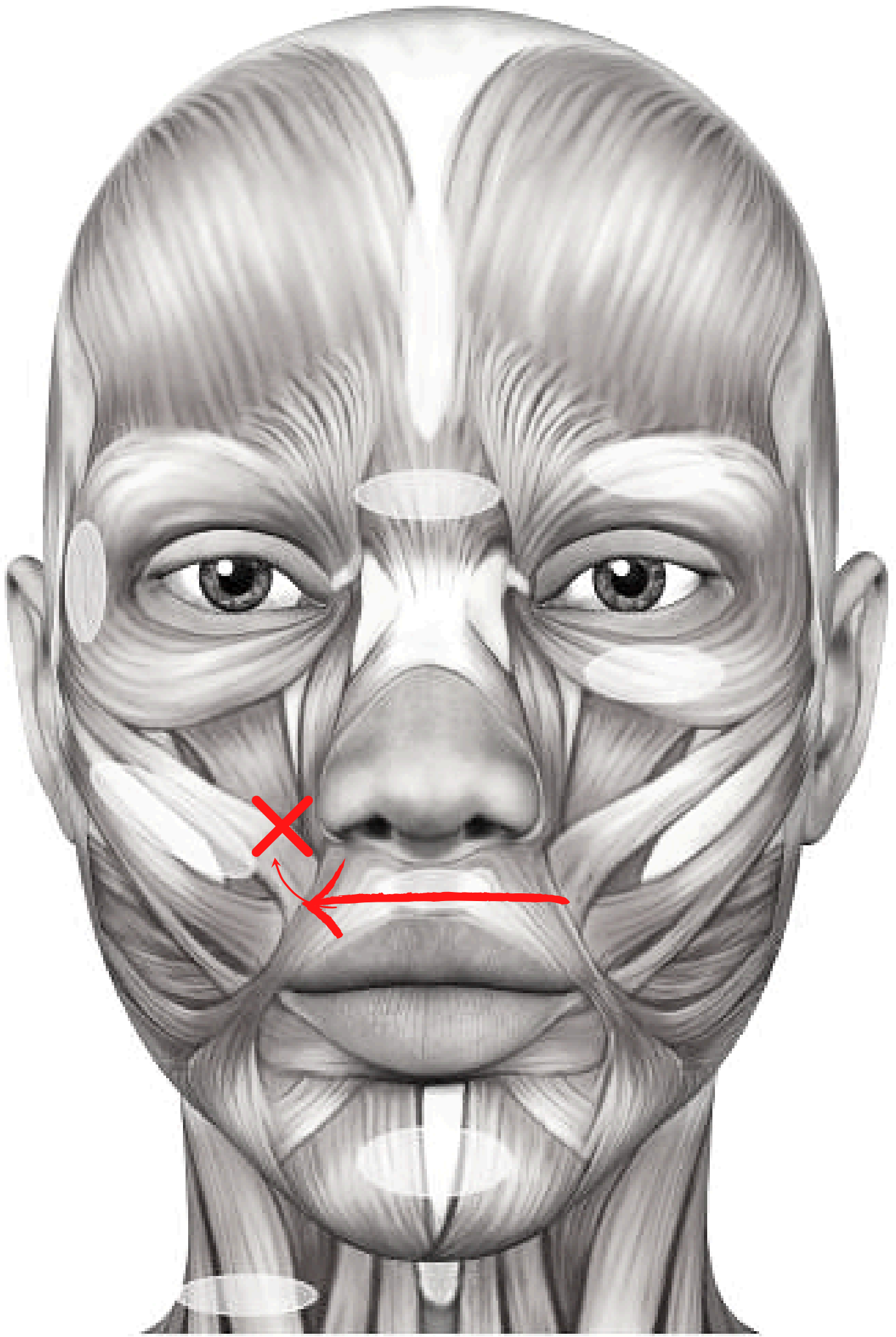


Passage 3.

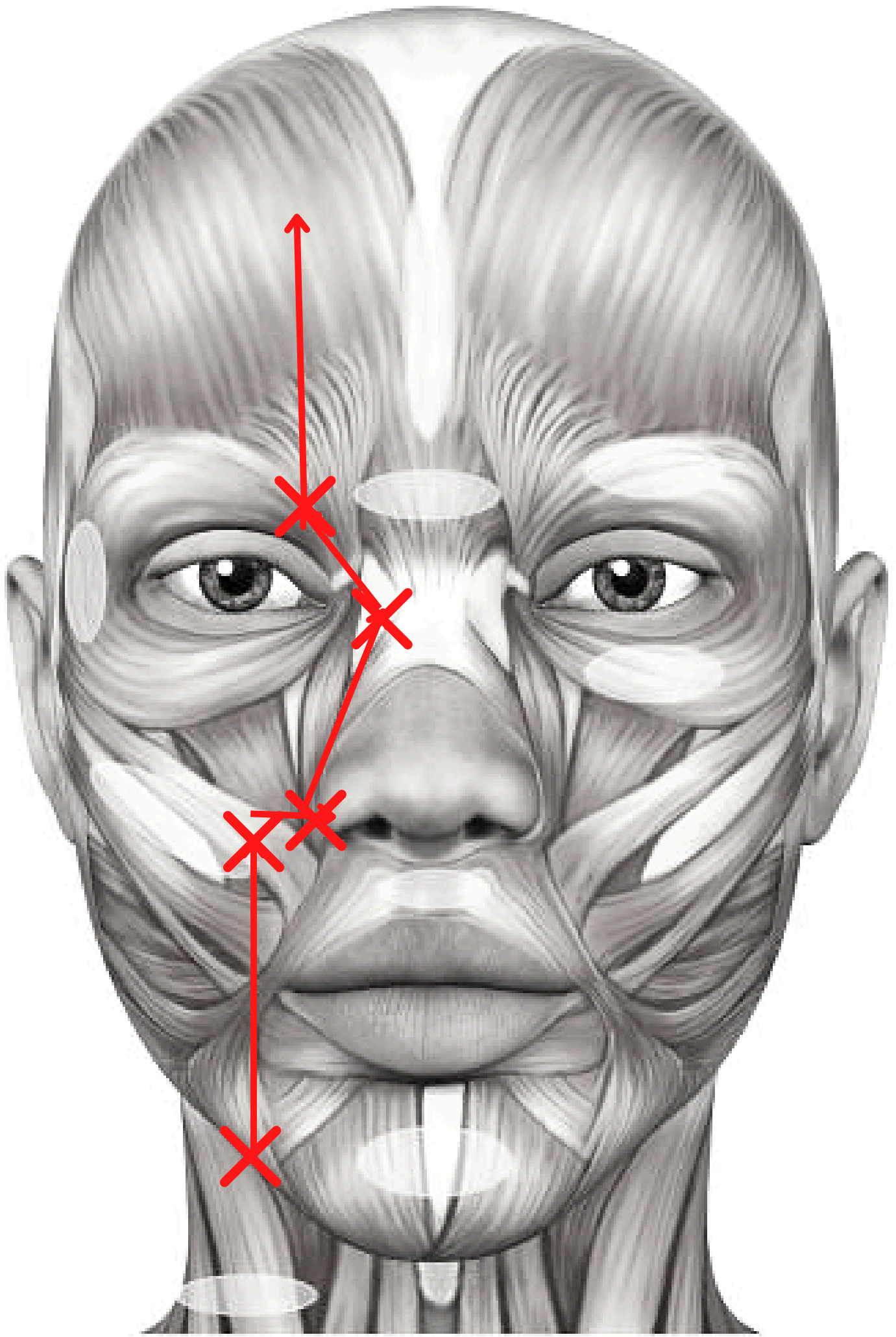


Passage 4.

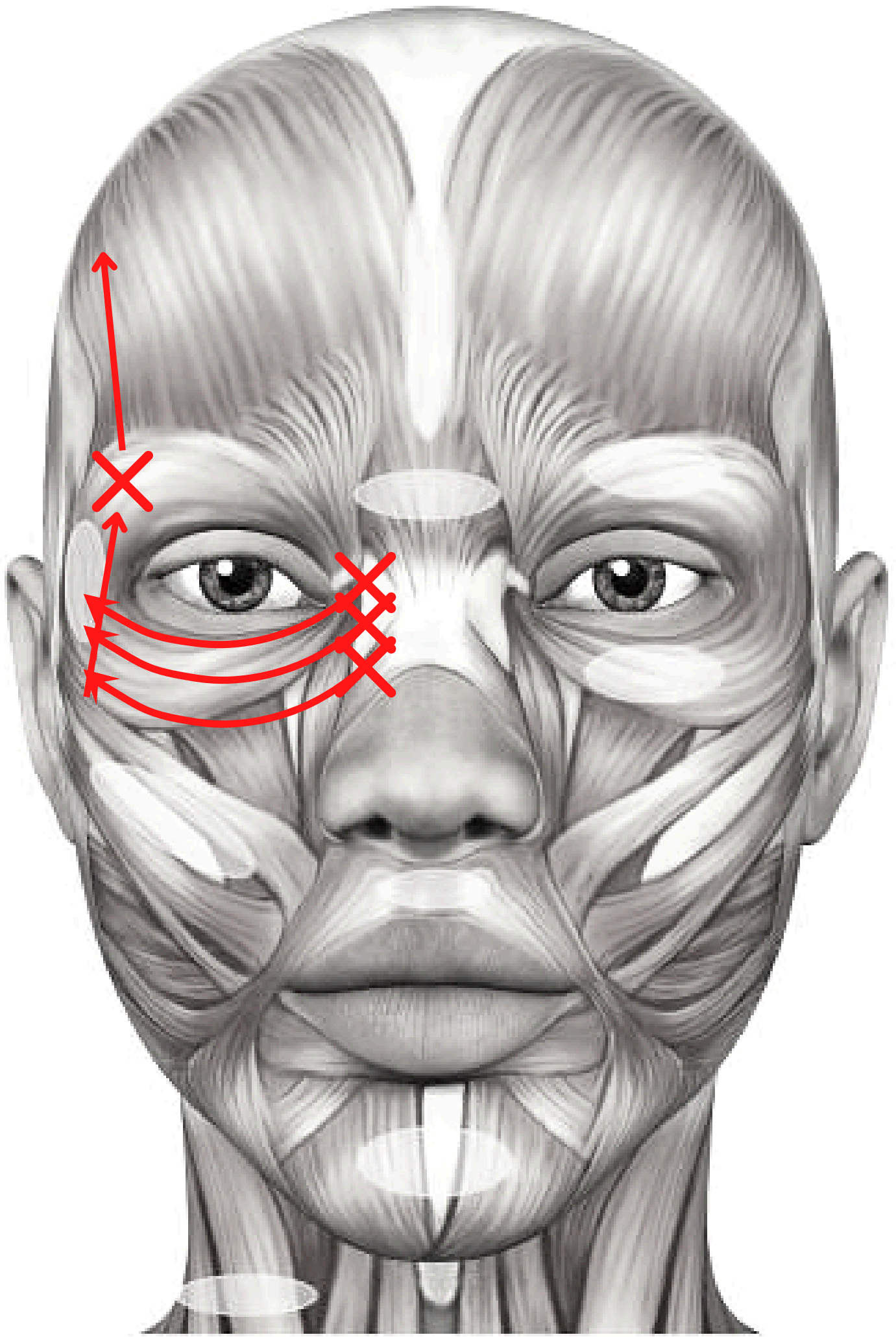

Passage 5.

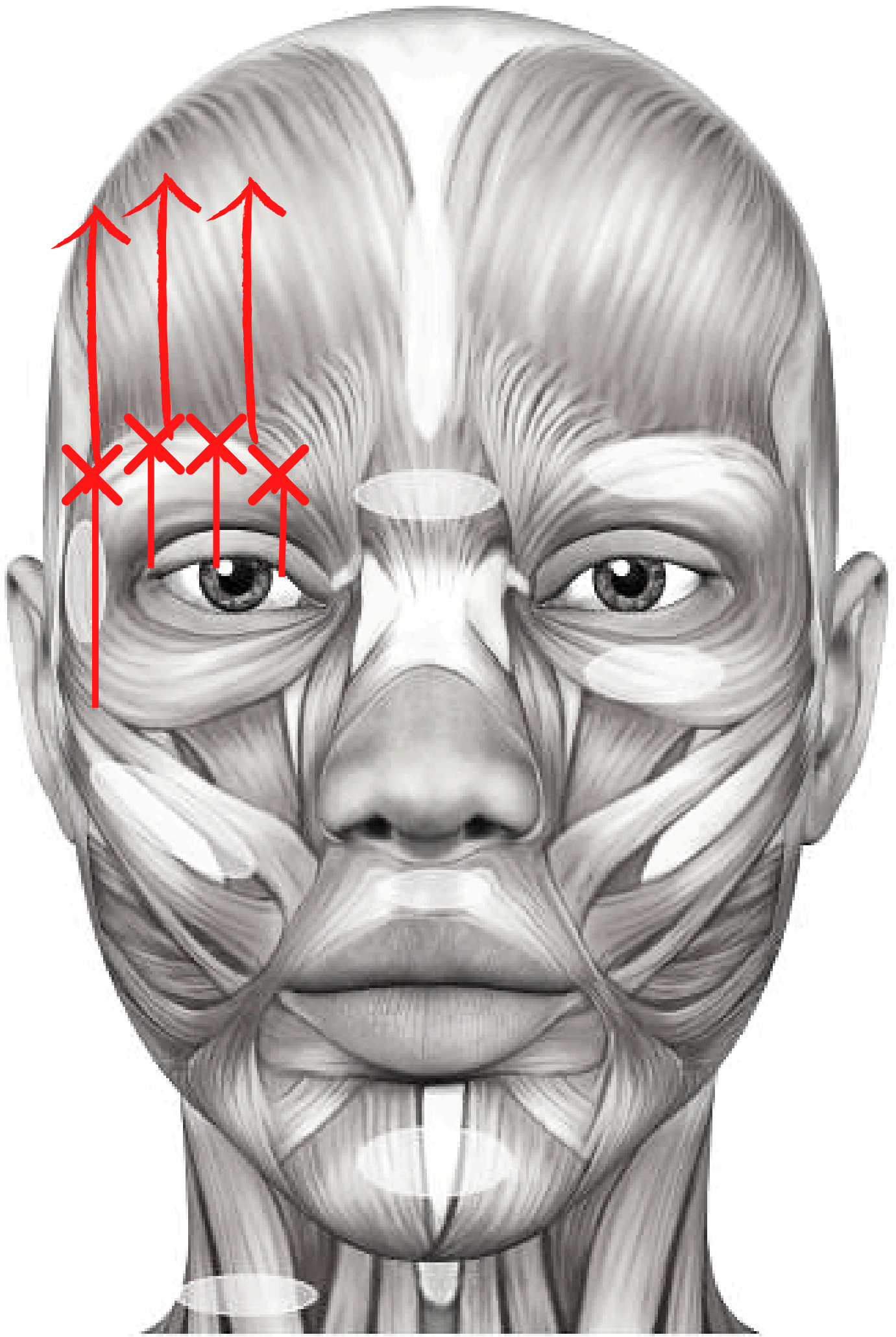


Passage 6.

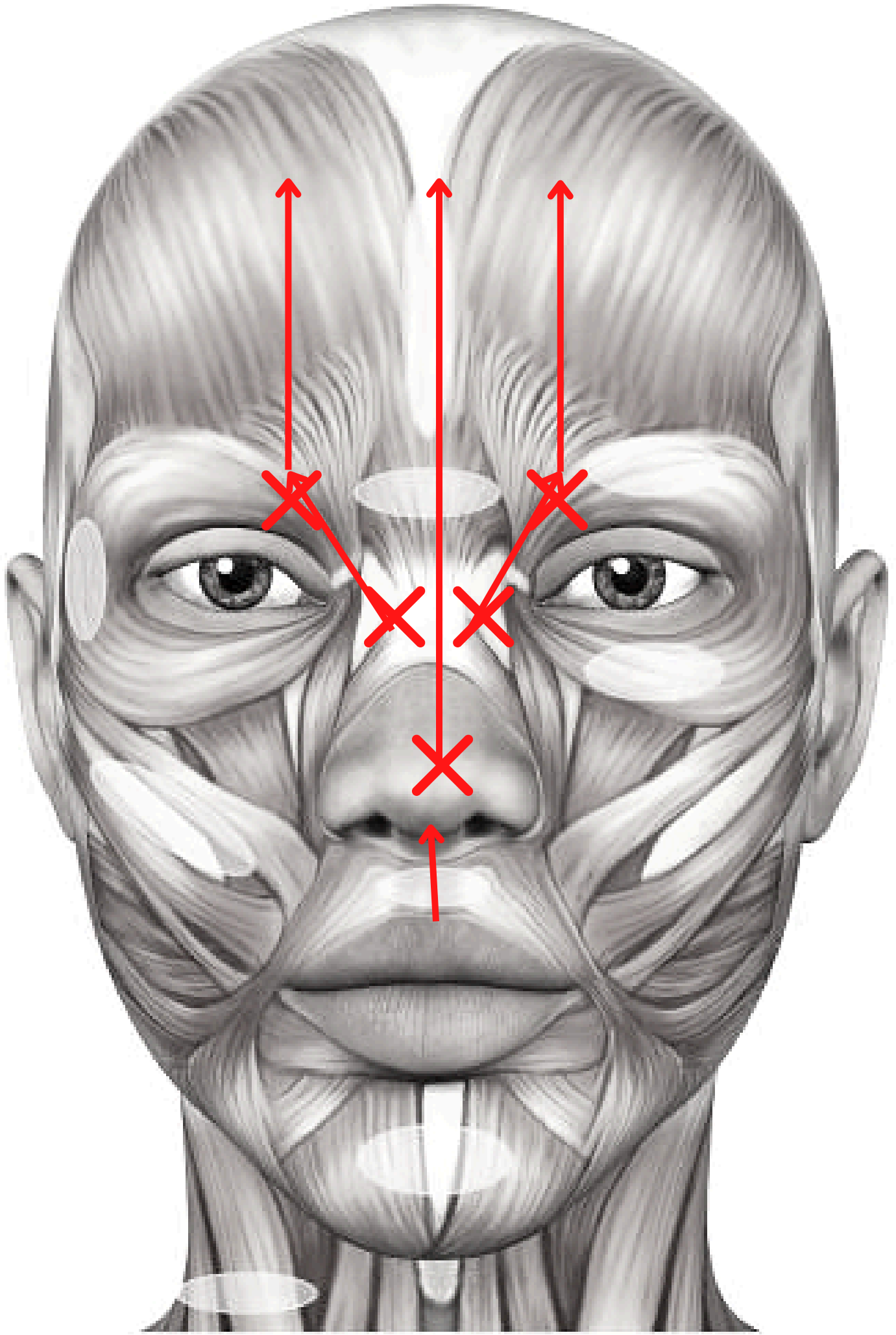

Passage 7.




Passage 8. REPEAT 1.

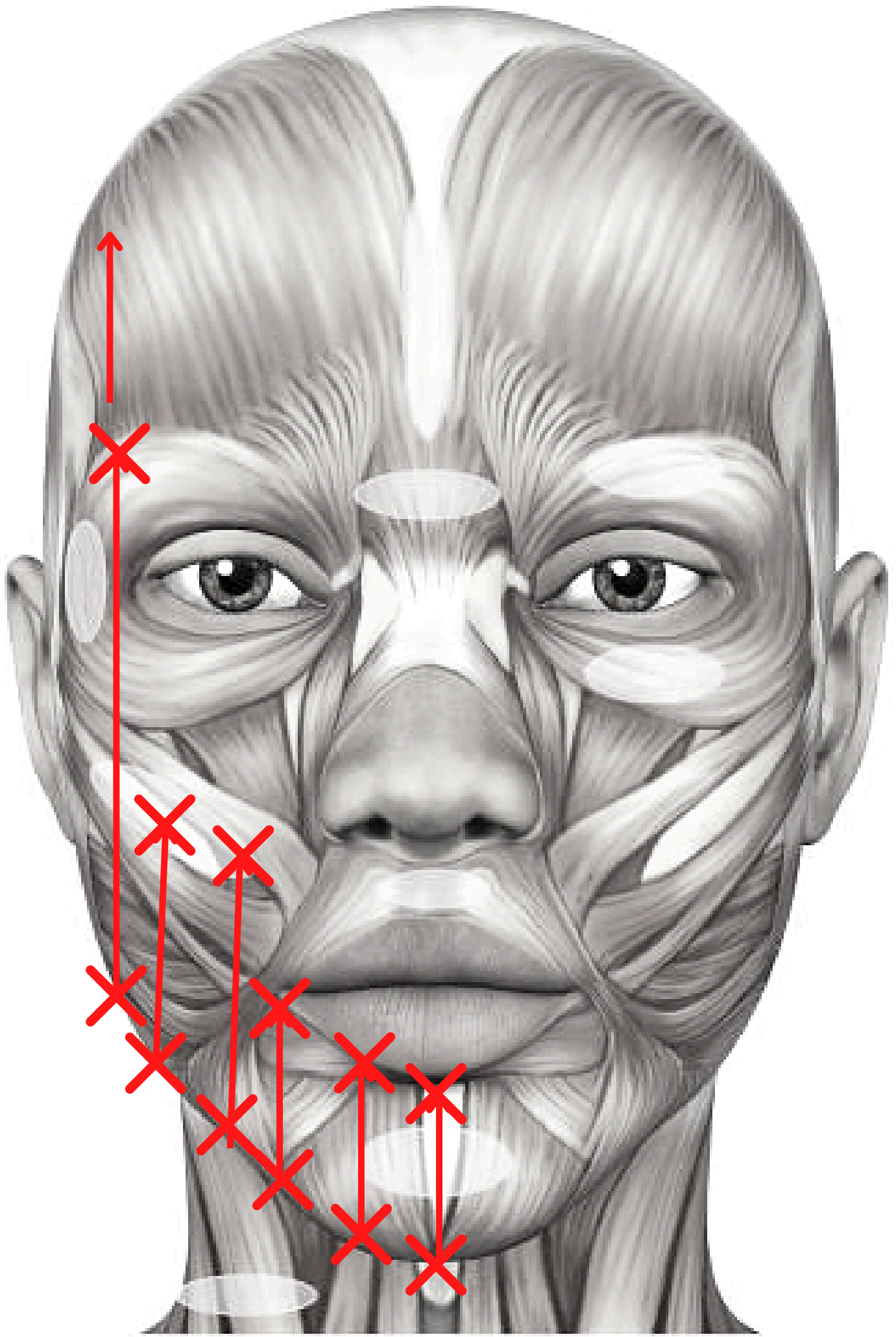

Neck muscles

Always include the treatment area first
with SuperBoost step 1 - Meso
Infusion.

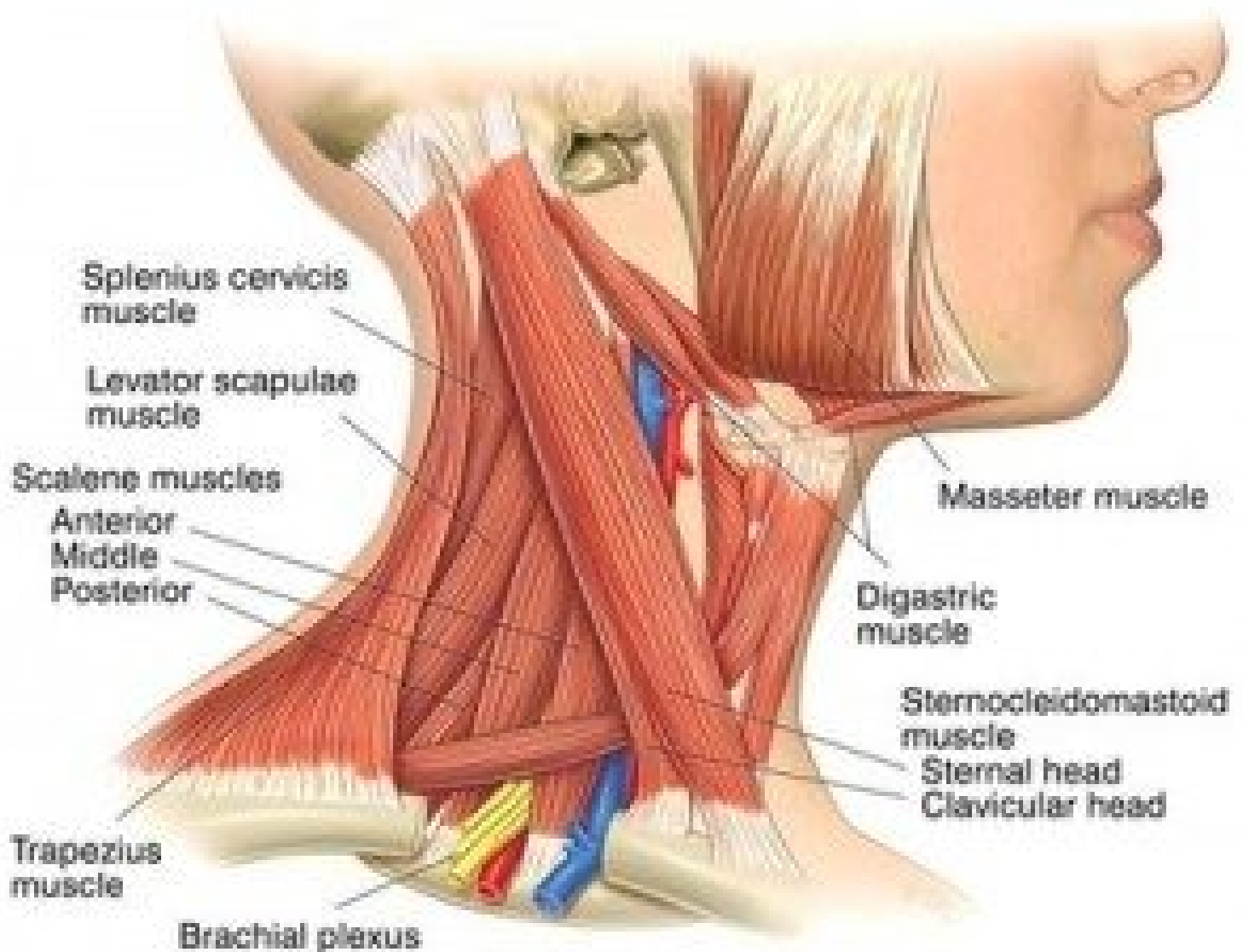

Step 1.

Turn on the device and wait a few seconds.



Step 2.

Click the top button twice to set the intensity.

2x




Step 3.

- Take enough gel and place the Biorefpen on the lower left side of the mouth.
- A red light will now illuminate.
- Go over the neck once and apply the gel. You must not treat the middle area of the neck due to the thyroid.



Step 4.

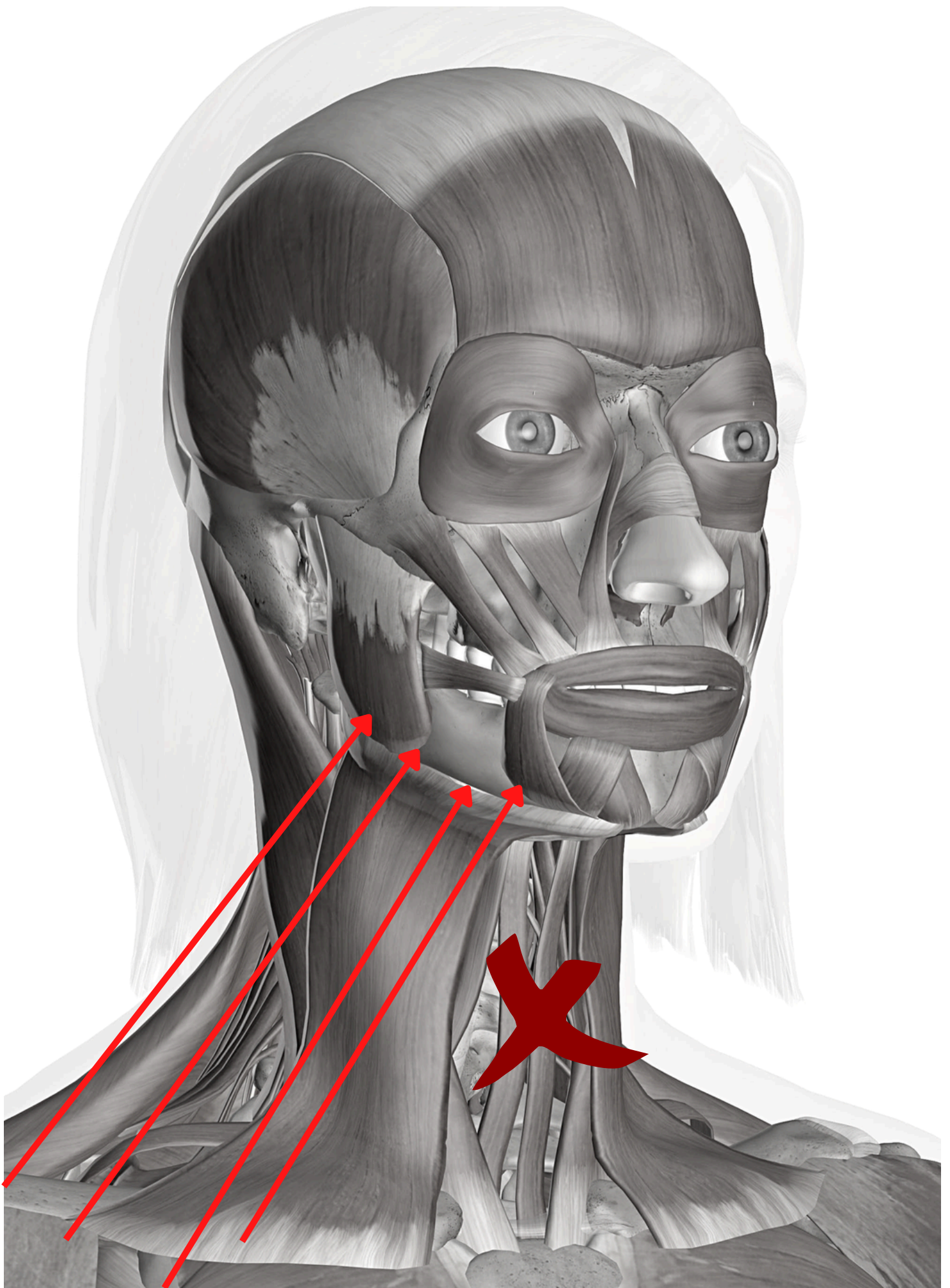


Step 5.

- Now click the bottom button twice.
- Take enough gel and put the Biorefpen at the bottom of the chin.
- A green and red light will now turn on alternately.
- We now follow the different trajectories from attachment to attachment. See drawing.


Always wait three sec at each attachment point and repeat each passage three times.



Step 6.

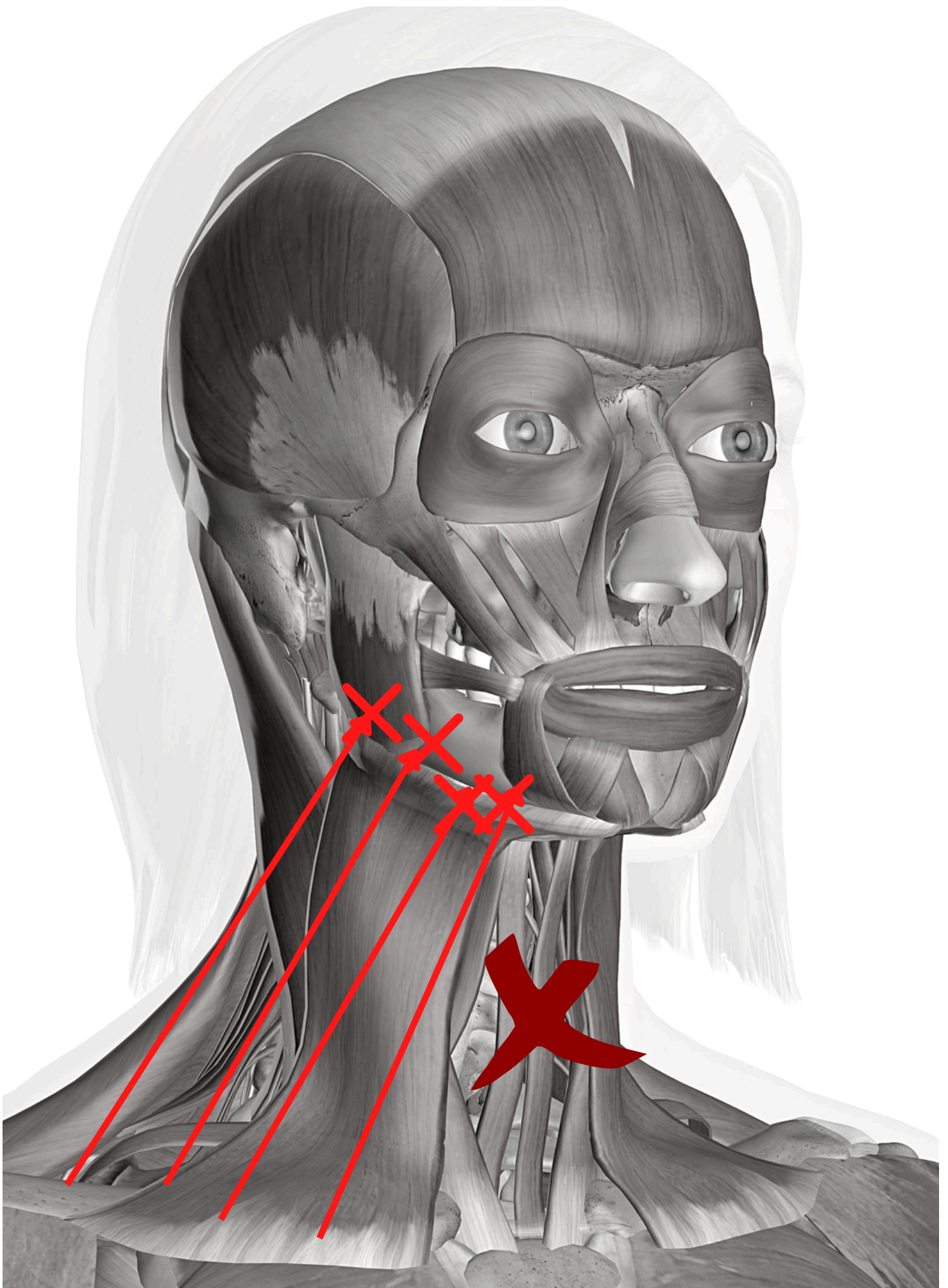

Step 7.

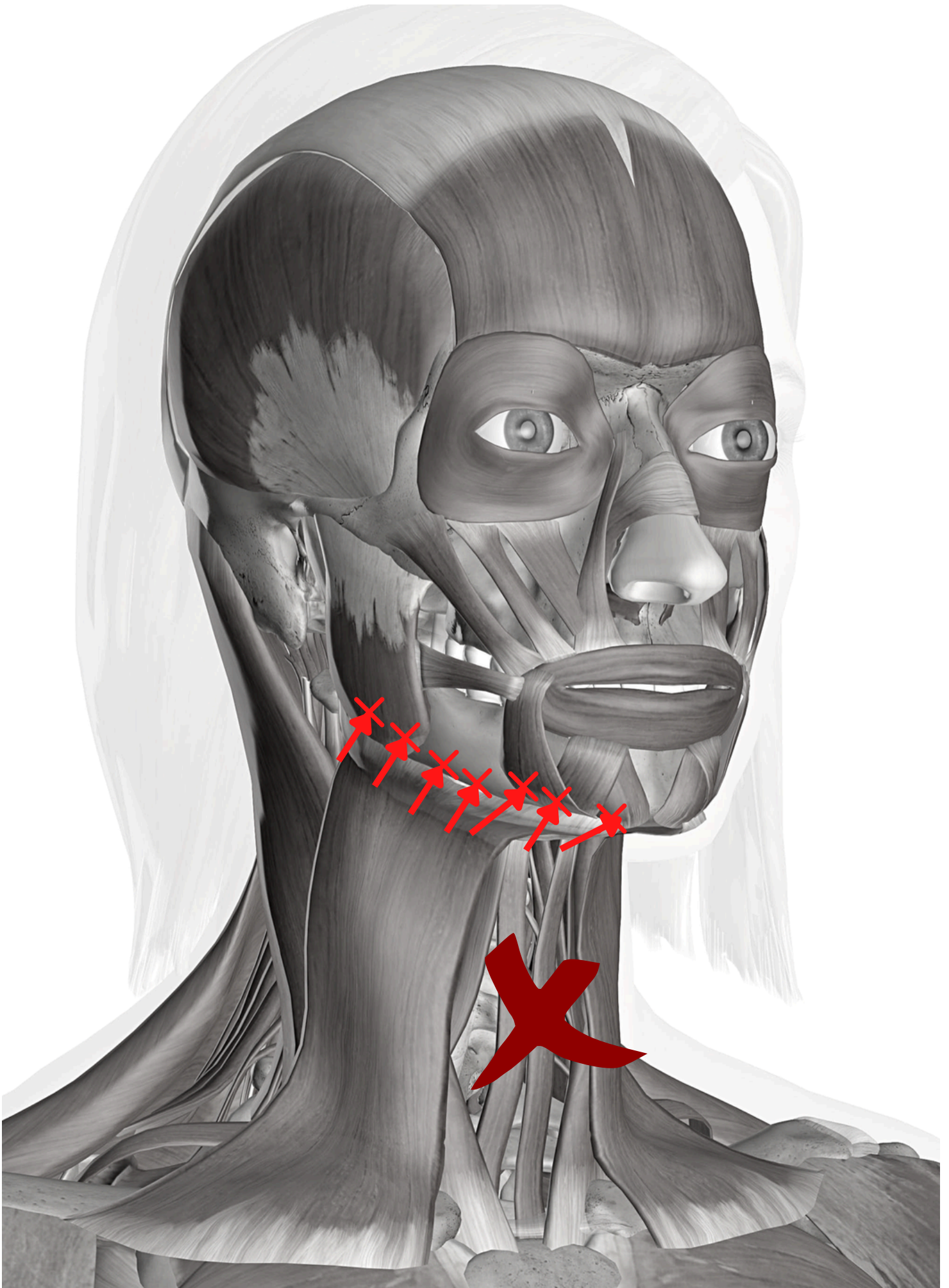

Step 8.

